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Class I Medical Device, according to the European directive 2017/745 on medical devices and as such complies with the essential requirements that apply to it, and must be used under the control of a medical specialist or orthopaedic technician to achieve efficacy in its therapeutic purpose and safety in its use, i.e. that its use does not represent any risk to the user's health.

To ensure the correct functioning and optimal performance of this medical device, read the instructions carefully before using it for the first time. Check the correct external appearance of the product and its integrity, as well as the integrity of the packaging.

Package contents: 1 belt, 3 fasteners: MAD55, 1 key: LAD55, 1 instruction manual, 1 advertising sheet.

INDICATIONS: Designed for patients requiring immobilisation. Facilitates postural changes providing controlled relative mobility. Also prevents manipulation by the patient and unauthorised persons.

CONTRAINDICATIONS: The product is contraindicated in case of patient agitation and for patients who may become detached from the device. Do not use in case of allergy to any of the components.

SAFETY INFORMATION: Do not use the ADIFIX positioning and restraint system without having read and understood the instructions. Any misuse of the system may result in the risk of serious injury or death. The safety of your patients depends on the correct installation and use of the ADIFIX system. The ADIFIX system must always be used by qualified healthcare personnel in accordance with the manufacturer's instructions and in compliance with the applicable state and EU regulations. Any private user must strictly comply with the doctor's orders. The condition of the product must be checked regularly. Any modifications made to the original product without the manufacturer's consent shall exempt the manufacturer from any liability arising from its use. This product should not be placed in direct contact with an injured area. Frequent checking and verification of the correct adjustment of the product on the user is recommended. Never leave the device within the reach of children and/or persons who cannot look after themselves. Due to the textile nature of the components, do not expose the product to a source of heat or moisture. Liquids cause the locks to fail; do not immerse the locks in any kind of liquid. In case the locking mechanism cannot be opened, the strips can be cut with scissors. Warning for pacemaker patients! A distance of at least 10 cm must always be maintained between the patient's pacemaker and the ADIFIX magnetic key, otherwise the patient may experience tachycardia. There are no known side effects. The ADIFIX standard restraint system must never be used without the lateral positioning straps, which prevent horizontal rotation of the patient in bed. Otherwise, the patient could fall over the side of the bed and strangle, risking death.

INSTALLATION INSTRUCTIONS:

- 1º Spread the belt over the bed and place the straps on both bed rails with a magnetic fastener on each side.
 - 2º Lay the patient on the belt and place the abdominal part with the magnetic clasp.
 - 3º Adjust the straps of the crossbar if necessary.
- Possibility of extension with the CAD56 extension (not included).

ADIFIX OPENING AND CLOSING:

Open the ADIFIX lock (MAD 55):

1. Place the magnetic key (LAD55) on the button on the top of the lock (MAD55).
2. Make sure that when opening the ADIFIX lock, the lock is not under pressure from the tapes - it may not open.
3. Lift the magnetic key together with the black button to open the lock.

Close the ADIFIX lock (MAD55):

1. Place the black button on the piston.
2. Check that the lock is locked. You should not be able to open the lock.

SIZE:

SIZE	CONTOUR
S	40-60 CM
M	60-100 CM
L	100-140 CM


COMPOSITION AD56: Canvas: 100% polyester. Bias: 100% cotton. Thread: 100% polyester. Eyelet and Washer: nickel plated brass. Buckle: nickel plated brass.

COMPOSITION AD56A: 100% polyester. Velour: 80% cotton, 20% polyamide Bias: 100% cotton. Yarn: 100% polyester. Eyelet and Washer: nickel plated brass. Buckle: nickel plated brass.


STORAGE AND STORAGE: Machine wash at 90° maximum. Do not dry clean or iron. Do not tumble dry. Do not bleach. The belts should be washed in washing machines that do not exert pressure on the metal parts of the system, otherwise these parts may deteriorate. Before each washing and drying, remove all ADIFIX fasteners in order to avoid damaging them and causing their failure. Dirty fasteners can be cleaned with a damp cloth. The seals must not be immersed in liquids of any kind.

Store the product in a dry place, protected from light, at room temperature in its original packaging.

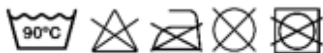AD56 :

AD56A :

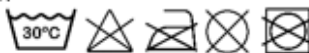REUSE: Given its therapeutic purposes, this product is not reusable by another person.

WASTE MANAGEMENT: As far as waste management is concerned, the product and its components comply with the provisions in force laid down by the local authorities.

WARRANTY: 24 months from the date of purchase. You must keep the invoice as proof of purchase for any claim. The product is intended exclusively for the use described in this manual. Any damage resulting from improper use is excluded from the warranty. Damages are limited to the value of the goods. Compensation for consequential damages is expressly excluded.

Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient are established.

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