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EN- Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0° or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all parts are tightened (handles, rubber tips...).
- ♥ Hold firmly and completely the handle when using.
- ♥ In case of falling : let the product fall down on the side, so that you don't fall on it.
- ♥ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- ♥ Don't use on wet or icy surfaces. (snow, ice, water..)!!
- ♥ Don't use in stairs, severe risk of injuries!!
- ♥ Use only as a walking aid and in proper way : don't use as a lever or to move furniture.
- ♥ ALWAYS observe the weight limit on the labelling of your stick.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- ♥ DO NOT hang anything on the Walking aid.
- ♥ In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.
- ♥ Be careful when using: beware holes and deformations in the floor.
- ♥ The suction cap must not be too worn down. If the cap shows considerable wear, ask your supplier or nursing staff to replace it.
- ♥ Any serious incident that has occurred in relation to the device shall be

reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

- ♥ The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. Intended use, indications and contraindications

Intended use:

Walking aids are used for two purposes: as part of a rehabilitation programme when the user is recovering from an injury or operation and as a long-term aid to mobility when the user has a permanent difficulty with walking.

These devices help in maintaining balance, aid in the prevention of injuries (particularly falls), and allow the person to attend to his/her errands independently. Mobility aids can be used indoor or outdoor, depending on the model.

Mobility aids are used to modify the weight redistribution: some of the weight carried through the legs when walking is transferred through the arms of the frame or stick as it is leant on for support. Products equipped with a seat allow short rest to the user.

Indications

- Loss of autonomy due to walking disorders - Balance and/or posture disorders
- Walking problems - Rehabilitation programme after an injury or an operation

Contraindication

Don't use in case of:

- Severe cognitive disorder - Severe loss of balance - Severe motor or perception disorders - Weakness in the upper limbs

3. Installation

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient.

Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

To use as a seat:

To unfold, push down each side of the seating stick until the 3 legs are in an horizontal position.

Don't use on a soft ground, one leg could sink into the ground and you could lose your balance.

To use as a stick:

Fold the seating stick.

Keep the seating stick close to your body while going to avoid accidents.

4. Cleaning

The stick has to be cleaned regularly with a mild cleaning agent and a soft cloth. NEVER use OILY substances!! Slip hazard!!

Ask your retailer or the technical staff to check the walking aid regularly.

The rubber tips must be checked and changed regularly.

5. Materials

Tube in aluminium covered with epoxy painting and polypropylene straps and seat.

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

This product has a 24 months warranty, from the date of purchase, against any manufacturing defect. Standards: CE. Biocompatible.

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